

MEDEX International, Inc.

User's Manual



This manual is for the following:

- 3T18 – 3S18 – 3A18
- 3T19 – 3S19 – 3A19
- 3T22 – 3S22 – 3A22
- 3T23 – 3S23 – 3A23

(Titanium, stainless steel, and aluminum rotatable pyramid adapters and receivers)

1- **APPLICATION:** This product is designed to be used for lower limb external prosthetic toe and knee direction adjustment applications. Before fitting this product onto the patient, all parts must be joined properly and all set screws must be secured enough for the patient to walk. In order to make final adjustments, there should be no loose screws.

2- **WEIGHT RATINGS:** The maximum patient weight ratings for each alloy of this product are:

3T19 – 3T22	Titanium	136kg/300lbs
3S19 – 3S22	Stainless steel	136kg/300lbs
3A19 – 3A22	Aluminum	75kg/165lbs

3- **TORQUE:** For permanent use, 8 mm SS set screws are 15 Nm
5-6 mm SS cap screws are 9 Nm

4- **WARRANTY:** All MEDEX Modular Adapters [Stainless Steel, Titanium, Aluminum] are unconditionally guaranteed for 2 years. Cutting, drilling, modifying or using the component other than described in this manual, will void the warranty.

5- **ATTENTION:**

A- Improper torque (uneven, insufficient or excessive) and installation may cause bolt or adapter failure.

B - Always select and pay attention to use quality *metric* tools.

6- **REQUIRED:** After the final adjustments, apply single drops of removable Loktite® [222R] for each of the set/cap screws and then torque the product.